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Dear patient,
Thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your physician or medical retailer.

Intended purpose

GenuTrain A3 is a medical device. It is a support for activation and stabilization in the case of complex knee problems such as osteoarthritis.

Indications

- Osteoarthritis of the knee and arthritis
- Chronic, post-traumatic and postoperative irritation
- Tendinopathy, ligament insertion degeneration and tendomyopathy
- Knee pain with functional instability due to general muscular imbalance
- Recurring joint effusion
- Prevention / relapse prevention
- Patellar lateralization

Risks of using this product

⚠ Important information

Please closely observe the specifications in these instructions for use and the instructions given by the professional*.

The benefits of this product are felt particularly during physical activity.

- Doff your support during longer periods of rest.
- Once you have been prescribed the GenuTrain A3, only use it according to your indication and any additional instructions given by a professional*. Please consult a professional² or your physician first if you want to use it together with other products. Do not modify the product yourself in any way; otherwise it may not help as expected or it may be harmful to your health. In these cases, any warranty or liability claims will be excluded.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, in rare cases, constrict the underlying blood vessels or nerves.
- If you notice any changes or an increase in symptoms while wearing the product, stop any further use and contact your physician.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a professional before first using the product. Restrictions may exist given the specific indication which the professional can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.

Caution: Upon doffing of the product, there is not sufficient support / sufficient protection of the body part requiring treatment. Any strain can cause additional damage to the affected body part, so avoid strain.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. This aid should be fitted and worn only after consulting your physician in the following clinical situations:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. In case of any raised scars with swelling, redness or excessive heat build-up
2. Varicose veins
3. Impaired sensation in the leg due to neurological abnormalities
4. Impaired circulation in the leg (e.g. diabetes mellitus)
5. Impaired lymphatic drainage and soft tissue swelling of unknown origin

Application instructions

Donning and doffing of the GenuTrain A3

Donning

1. Firmly grip the support at the top of the lateral stays on both sides (donning aid with thumb guidance) **1** and pull it over the knee joint until the kneecap is surrounded by the pad. **2 – 4**
Note that the support can only be used on one side (left or right). The long side of the pad should be positioned on the inside of the knee. In case of doubt, the sewn-in label indicates the correct side.
2. If required, correct the fit of the support.
3. The support must not be put on using donning aids that belong to other aids, as this can destroy the silicone coating.

Doffing

To doff the support, pull it down at the bottom of the support near the stays. **5**

Cleaning instructions

Please use mild detergent and a mesh bag. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees the ideal effectiveness of the product.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims.

Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain A3 have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the professional were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a professional*. Before first using our medical device, please seek the advice of a physician or professional* as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this professional* as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the professional*, please contact your physician or distributor, or contact us directly.

Duty to report

Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Silicone (SI), Polyamide (PA), Thermoplastic polyurethane (TPU), Elastane (EL), Cotton (CO), Thermoplastic elastomer (TPS-SEBS), Polyurethane (PUR), Polyester (PES)

 – Medical device

 – DataMatrix Unique Device Identifier UDI

* A professional is any person who is authorized according to the state or institutional regulations as well as other provisions relevant to them for fitting and instruction in the use of orthopedic aids. As the manufacturer, we strongly recommend that this person be trained in orthopedics or possess comparable acquired competencies as well as dispose of orthopedic infrastructure.

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